

85x75x175MM

MediGenix
ever better

SPACER WITH ADULT MASK

MGX-097

MediGenix
ever better




Instructions for use (single patient use only)




1. Look inside both ends of the spacer for any loose debris.




2. Fit the mask connector securely onto the spacer's mouthpiece. Shake the inhaler immediately before use as per the instructions supplied with it.

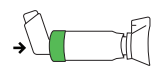

3. Remove the cap from the inhaler and insert the inhaler firmly into the base of the spacer.

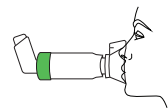

4. Exhale completely. Place the mask over your mouth and push it against your face to ensure an effective seal.

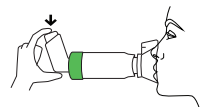

5. Press the inhaler once and inhale slowly and deeply until a full breath has been taken. If possible, hold your breath for 5-10 seconds before exhaling. Repeat for 5-6 breaths, whilst holding the mask in place.

NB: Follow the instructions supplied with your inhaler regarding how long you should wait before repeating the above steps.


NB: Slow down if you hear the whistle sound because it means you are inhaling too quickly.

NB: The spacer should be replaced after 12 months of use.

MediGenix
ever better




Cleaning instructions (clean once a week)

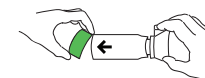

1. Remove the base of the spacer and twist the mask section to disassemble the device.




2. Soak all the parts in lukewarm, soapy water (never hot or boiling). Gently agitate the water.

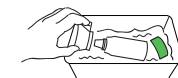

3. Rinse in clean water.



4. Shake parts to remove excess water and leave to air dry in a vertical position. Do not use a cloth to dry the parts.

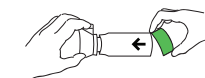

5. Once completely dry, reassemble the device and store it safely in a clean, dry place.

NB: Dishwasher safe on the top shelf at temperatures below 70°C. Do not wash with very dirty dishes.

Syringa UK Ltd
Rival House Daux Rd
Billingshurst
RH14 9SJ
United Kingdom
+44 (0) 1403 289370
sales@syringa-uk.co.uk
customercare@syringa-uk.co.uk

UK REP

Syringa UK Ltd
Rival House Daux Rd
Billingshurst
RH14 9SJ
United Kingdom

EC REP

Rep Europe GmbH
Annstr. 9 B
64347
Griesheim
Germany

ISO 9001
ISO 13485



SPACER WITH ADULT MASK

MGX-097

5+ yrs



5 060796 620566

SPACER WITH ADULT MASK

MGX-097

5+ yrs